



## How to Use

### PREPARE

The BladeCASSETTE:

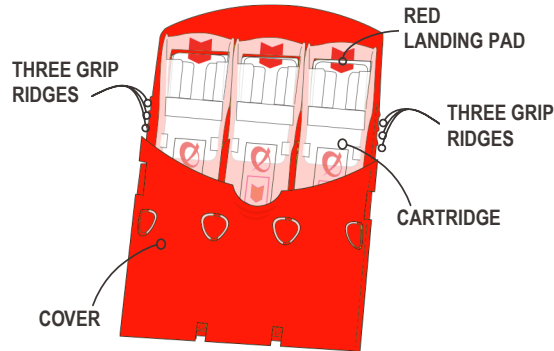

1. Squeeze the grip ridges on both sides of BladeCASSETTE to release the cover.

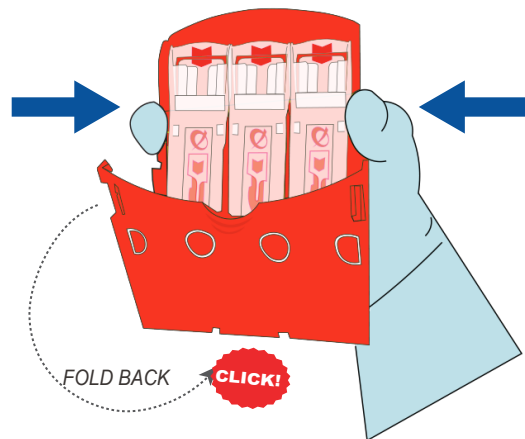


2. Fold back the cover until you hear a click locking it in place, then stand the BladeCASSETTE upright on a drape.


### REMOVE BLADE

Single-Handed Action:

3. Using one hand, slide the scalpel tip into the cartridge:

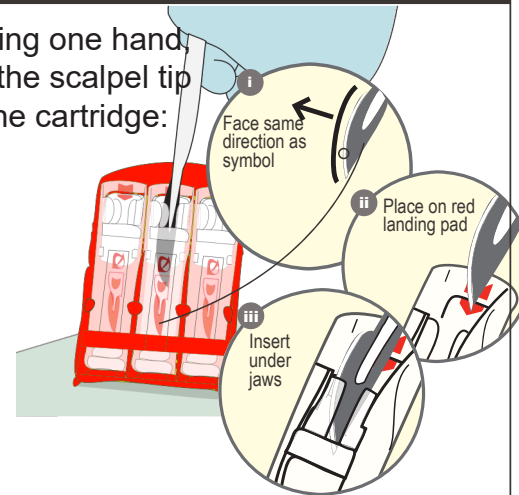

4. Tilt BladeCASSETTE up to 90°, then push down on scalpel handle until you hear a click.

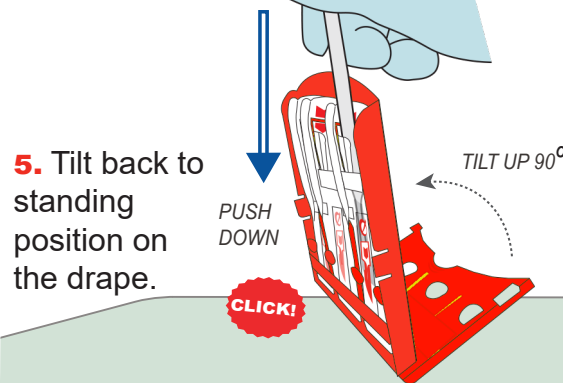


5. Tilt back to standing position on the drape.

### SAFE


To Use Two Hands:

6. Brace the BladeCASSETTE with your free hand and pull out scalpel handle.

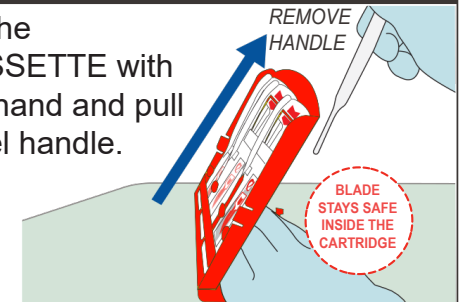


7. Repeat blade removal process as necessary to remove up to 3 scalpel blades.

8. Close the cover, back to the starting position.

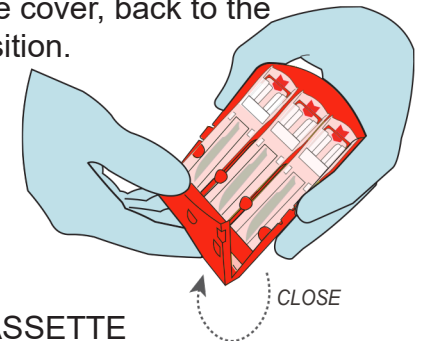

9. BladeCASSETTE is ready for disposal in sharps bin.

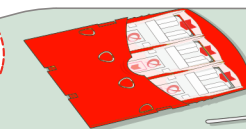

Scan the QR Code to see the BladeCASSETTE in action.

[https://www.youtube.com/watch?v=OwMiBjvY\\_ZQ](https://www.youtube.com/watch?v=OwMiBjvY_ZQ)